

CMPT Case History Sheet, M213

shipping date: **Nov 1, 2021** report due date: **November 16, 2021**

CMPT Specimen No.	Sample Type (simulated) see Note 1 below	History Description	Instructions Set up and report as per your laboratory protocol.
PC213	Paper Challenge	See PC213 questionnaire	
G213 (companion to M213-4)	Leg Wound smear	38 year old car crash victim with leg wound	DO NOT heat or alcohol fix the smear prior to staining. Please stain smear <u>label side up</u> . Stain the slide using Gram stain reagents and store the slide for future reference. Perform this challenge if your laboratory does any gram staining. If your laboratory does not routinely read this type of smear, note this with your results. To enter results on-line, use the member.cmpt.ca .
M213-1	Nephrostomy Tube Urine sample	48 year old with renal stones	To enter results on-line, please go to member.cmpt.ca Susceptibility required – Ampicillin, Vancomycin, Ciprofloxacin
M213-2	Stool sample	70 year old traveller	To enter results on-line, please go to member.cmpt.ca Susceptibility required – Ampicillin, Ciprofloxacin, SXT
M213-3	Bronchoalveolar Lavage sample	45 year old in-patient with lymphoma and pneumonia	To enter results on-line, please go to member.cmpt.ca
M213-4 (companion to G213)	Leg Wound swab	38 year old car crash victim with leg wound	To enter results on-line, please go to member.cmpt.ca
M213-5	Peritoneal Fluid sample	55 year old patient with central line catheter	To enter results on-line, please go to member.cmpt.ca
<p><i>Note 1: CMPT recognizes that some of these samples may not be routinely accepted in their current format. However, these formats are currently used for proficiency testing samples only. Please process samples in this format, if the sample type is part of your laboratory's protocol.</i></p> <p><i>Note 2: If your laboratory normally performs susceptibility testing on an isolate and there is no requirement designated, CMPT does not require a susceptibility report for the challenge. If you would like to report susceptibilities, please enter the results in the comments box of the Specimen Identification form or e-mail the report to cmpt.path@ubc.ca.</i></p>			

If you would like to fax your report to CMPT, report forms (Gram Smear, Urine, Clinical Relevancy, Specimen Identification, Susceptibility) can be printed from www.cmpt.ca.

Reporting Results

- If relevant, state if the organism would be sent to a reference laboratory for complete identification.
- Interpretation of laboratory response is based on organism identification or laboratory phrase.
- For conventional or "classical" testing, include pertinent individual tests and results.
- If a simulated proficiency sample is a **specific specimen** that your laboratory would **normally perform a gram smear**, it is advised to do the same for a CMPT swab. Currently CMPT swabs do not contain cells
- If a request accompanies a sample to report "**as per your laboratory protocol**", include only susceptibility results that would be included in the final clinical report. If antimicrobial testing is not applicable in your laboratory, please note this. Interpretations of susceptibility values **must** accompany susceptibility results **R, I, S**.
- Samples designated to be assessed based on **clinical relevancy**; grades will be based on the exact wording a laboratory would include in a final clinical report
- Fill in the information requested on all the report forms as completely as possible.

Please submit a report even if your laboratory does not normally perform testing on a particular type of specimen and indicate accordingly on the report. If a report for a specimen challenge is not submitted, it will be entered as "no report" for the challenge.

Important:

- Follow the instructions enclosed (available at www.cmpt.ca) and use only the forms specified for the corresponding challenge. If the wrong form is used, the challenge will be graded as unacceptable (0).
- Use only CMPT identifiers when reporting results. All identifiers can be found on the case history sheet included with your package. The challenge will be graded as unacceptable (0), if identifiers used are not recognized by CMPT.
- Report 2 CMPT identifiers when reporting results. Use the CMPT challenge number and the source and specimen type. Also, use full CMPT identifiers, eg. G213, GS213-1, M213-1, when reporting results. The challenge will be graded with a reporting error (RE) point for each component of a challenge where the identifier is absent, partial/incomplete or incorrect. RE points will not be awarded to ungraded challenges or challenge components.+
- Do not use acronyms when reporting bacteria in gram smears and/or culture. Reporting acronyms for bacteria will be graded as unacceptable.

Report forms can be obtained from the web site, www.cmpt.ca or phone by 604-827-1754 or 1-866-579-CMPT (2678).

Results should be entered via the on-line data entry program, www.cmpt.ca (Please see enclosed instructions)

If you have any questions, please contact CMPT as soon as possible